
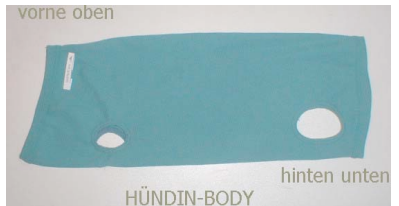
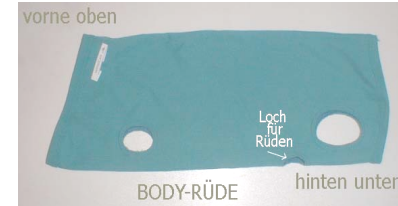
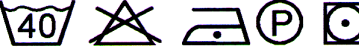
	<p><b>CROVETCARE</b>          TELEFON BROJ: 042 331-895          ili E-MAILOM: <a href="mailto:dr.lukman@vz.t-com.hr">dr.lukman@vz.t-com.hr</a>          INTERNET: <a href="http://www.dr-lukman.hr">www.dr-lukman.hr</a></p> <p>Ako imate pitanja, molimo kontaktirajte nas!          Telefon: 00385 42 331-895</p>	<p style="text-align: right;"><b>Dr. Norbert Freitag</b>  <b>VetMedCare</b> </p> <hr/> <p style="text-align: center;"><i>Innovative Pet Supplies</i></p> <p><b>No.IVA.Eur.: ATU 35774408</b></p> <p style="text-align: right;">AT-6973 Höchst • Hauptstr. 35c          Telefono: +43-5578-76648          Fax: +43-5578-74299          E-mail: <a href="mailto:vetmedcare@vetmedcare.at">vetmedcare@vetmedcare.at</a>          Internet: <a href="http://www.vetmedcare.at">www.vetmedcare.at</a></p>
--	--	---

Predmeta br.	Predmeta	Veličina	<b>VELIČINE</b>	Neck mjerenje u cm	Grudi mjerenje u cm	Udaljenost rijeda / stražnje noge u cm
<b>ženske – No.Art. 10.008– 10.016</b>			<b>muški – No. Art. 10.021 – 10.026</b>			
10.008	Dog&cat-body	XXXS Super mini	Mačići, Malen Psi,Zamorac, Yorkshire Terrier, Lhasa, Chihuahua,	8 - 14	10 - 15	12
10.009	Dog&cat-body	XXS mini	Mačići, Malen Psi,Zamorac, Yorkshire Terrier, Lhasa, Chihuahua,	10 – 15	15 – 20	16
10.011 10.021	Dog&cat-body	XS	Yorkshire Terrier, Lhasa, Chihuahua,	20 – 25	30 – 40	38
10.012 10.022	Dog&cat-body	S	Maltezer, Patuljasti jazavčar, pudl ili špic, Terrier, Bichon	25 – 35	40 – 50	46
10.013 10.023	Dog&cat-body	M	Jazavčar, Foxterrier, Jack Russell Terrier, Mops, Pudel, Beagle	35 – 45	45 – 60	50
10.014 10.024	Dog&cat-body	L	Collie, Chow Chow, Gubičar, Cocker Spaniel	45 – 55	60 – 75	60
10.015 10.025	Dog&cat-body	XL	Njemački ovčar, Doberman, Dalmatiner, Zlatni retriever,	55 – 60	60 – 90	67
10.016 10.026	Dog&cat-body	XXL	Bernardinac, Mastiff, Rottweiler, Doga	20 – 25	30 – 40	38

**USEFUL INFORMATION**

- Every **dog&cat bodysuit** has a size label (XXXS super mini, XXS mini, XS, S, M, L, XL, XXL) at the **top-front-inside**.
- First, reverse the **dog&cat bodysuit** over your hand until the end of the **dog&cat bodysuit** is at your fingertips. Then slip it over the dog or cat beginning from the head and pull the front legs and the hind legs through. For males, the penis must also be positioned at the respective opening.
- Smooth out the **dog&cat bodysuit**. **Done!**
- Determining the size:** The **distance between the front and hind legs** as well as the chest measurement provide a quick rough idea. Moreover, the listed breed names may be of help.
- As the high-quality materials **elastane and polyamide** are extremely elastic both horizontally and vertically ways, the **dog&cat bodysuit should be a little bit smaller than the dog or cat for tight fit**. Moreover, for female dogs, the stretch effect of the **dog&cat bodysuit** results in a stronger curvature underneath at the back, making it easier for the dog to urinate.

 <p style="text-align: center;">HÜNDIN-BODY</p>	 <p style="text-align: center;">BODY-RÜDE</p>	<p><b>Upute za pranje:</b>  Visokokvalitetni materijali ELASTAN i POLYAMID daju vrhunsku elastičnost, kako po dužini, tako i po širini. <b>(90/10) PA/EL</b></p> <p>Mi vam želimo puno zadovoljstva s ovim vrijednim proizvodom. U svezi dodatnih pitanja molimo da nam se obratite telefonski ili faxom na broj 042 331-895 ili e-mailom: <a href="mailto:dr.lukman@vz.-t-com.hr">dr.lukman@vz.-t-com.hr</a>          Vaš CROVETCARE TEAM!</p>
--	--	--