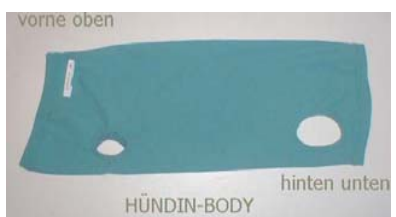
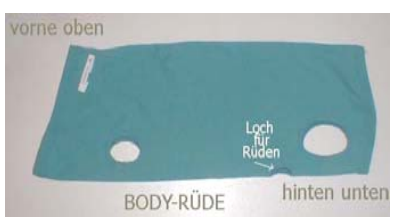



	ORDER FAX: +43-5578-74299 or INTERNET: www.vetmedcare.at	Dr. Norbert Freitag VetMedCare 	Hauptstr. 35c A-6973 Höchst
		<i>Innovative Pet Supplies</i>	Telefono: +43-5578-76648 Fax: +43-5578-74299 E-mail: vetmedcare@vetmedcare.at Internet: www.vetmedcare.at
No. IVA.Eur.: ATU 35774408			

Item no.	Item	Size	Usage Guide - Size Table	Neck measurement in cm	Chest measurement in cm	Distance front/hind legs in cm
FEMALE – No.Art. 10.011 – 10.016 – Indicated by a red point or female			MALE – No. Art. 10.021 – 10.026 – Indicated by a blue point or male			
10.012 10.022	Dog&cat-body	XS	small Yorkshire Terrier, small Lhasa, Chichuahua, Pekingese	15-20	20-30	38
10.013 10.023	Dog&cat-body	S	Maltese, Miniature Poodle, Spitz, Terrier, Bichon	20-25	30-40	46
10.014 10.024	Dog&cat-body	M	Dachshund, Fox Terrier, Jack Russell Terrier, Mops, Beagle	25-35	40-50	50
10.015 10.025	Dog&cat-body	L	Collie, Appenzeller, Chow Chow, Schnauzer, Cocker Spaniel	35-45	45-60	60
10.016 10.026	Dog&cat-body	XL	Alsatian, Doberman, Dalmatian, Golden Retriever, Doggue	45-55	60-75	67
10.012 10.022	Dog&cat-body	XXL	St. Bernard, Mastiff, tall Alsatian, Rotweiler, Danish Doggue	55-60	65-90	38

USEFUL INFORMATION

- Every **dog&cat bodysuit** has a size label (XS, S, M, L, XL, XXL) at the **top-front-inside**.
- First, reverse the **dog&cat bodysuit** over your hand until the end of the **dog&cat bodysuit** is at your fingertips. Then slip it over the dog or cat beginning from the head and pull the front legs and the hind legs through. For males, the penis must also be positioned at the respective opening.
- Smooth out the **dog&cat bodysuit**. **Done!**
- Determining the size:** The **distance between the front and hind legs** as well as the chest measurement provide a quick rough idea. Moreover, the listed breed names may be of help.
- As the high-quality materials **elastane and polyamide** are extremely elastic both horizontally and vertically ways, the **dog&cat bodysuit should be a little bit smaller than the dog or cat for tight fit**. Moreover, for female dogs, the stretch effect of the **dog&cat bodysuit** results in a stronger curvature underneath at the back, making it easier for the dog to urinate.

		Care information:  Material: 90 % Polyamide 10 % Elastane (90/10) PA/EL
We wish you a lot of fun with this high-quality product. Please do not hesitate to contact us by FAX or MAIL if you have any questions. Your VetMedCare TEAM!		